




HANDOUT


How to collect and transfer blood using a pipette

- The blood collection should be done from the finger in palm-up position.
- Gently squeeze the bulb of the pipette and touch the tip to the drop of blood.
- Then gently release the bulb to draw the blood up to the first line on the pipette.
- To transfer the blood, touch its nozzle to sample window on the test tray (square hole with letter A) and gently squeeze the bulb.
- The blood needs to reach and be absorbed by the pad at the base of the window. If the blood is mostly deposited on the plastic edges of the window, but does not reach the pad, the test will not work correctly.


TOO LITTLE
BLOOD

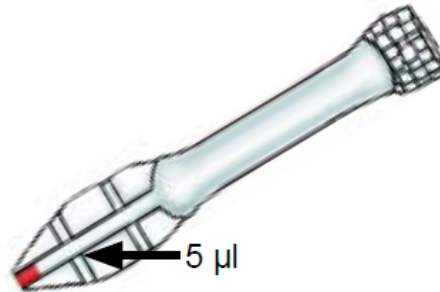

CORRECT
AMOUNT
OF BLOOD

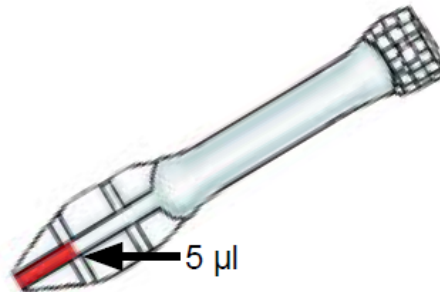

TOO MUCH
BLOOD

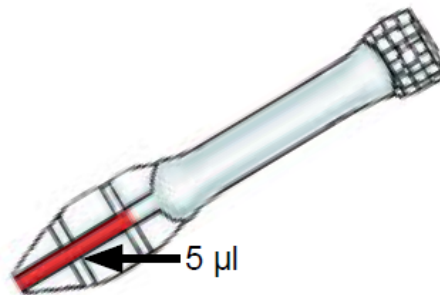

Source: CHAI/iCCM Private Provider Training for Child Health Program in Uganda, Day 2